**GR EN**

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## PRODUCT DESCRIPTION

- ❖ The air compression therapy system is one way of delivering high quality lymphatic decoupling. It improves blood and lymph circulation by preventing deep lymphatic thrombosis (DVT) and pulmonary embolism (PE), blain, and providing relief to lump.
- ❖ It contributes to the circulation of blood and lymph in arms, feet & waist and boosts muscle tissue pressure in order to remove the lump or pain
- ❖ Also it helps to enhance the physical functions of people with disabilities.
- ❖ It is used for the prevention and treatment of various diseases related to blood circulation, upon consultation with the appropriate doctor.
- ❖ It is used for people with diabetes, with post-thrombotic syndrome upon consultation with the appropriate doctor.
- ❖ Finally, for athletes, it enhances muscle relaxation before training. Moreover it is recommended to be used after training because it helps to restore muscle from the accumulation of lactic acid.
- ❖ It is important to be used on medical prescription and by qualified and technical or nursing staff.

## CAUTIONS

- ❖ If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions please contact with the dealer or the proper technical person before you use the device because may be caused damage.
- ❖ Consult the doctor or therapist to determine the correct setting and use of the device.
- ❖ Keep away the device from heat sources.
- ❖ The time limit of the device is determined by the wear of individuals parts.
- ❖ Do not allow the children to play or to use the pump.


## INTENDED USE

The air compression therapy system is indicated for use by medical professionals and patient at home, who are under medical supervision, reducing swelling and preventing thrombosis in the lower extremities or treatment of truncal or arm breast cancer-related lymphedema. Such as Primary lymphedema, edema following trauma and sport injures, post-immobilization edema, Venous insufficiencies, Lymphedema.

The system can be used at home or in hospital, to patients who are over 18 years old.

## OPERATION USE

- ❖ **Before you use the device, you have to open the cover in the lower surface and remove the safety screw. This screw is used ONLY to protect the device during the transfer.**
- ❖ Connect the air supply cable to the pump inlet
- ❖ Connect the pump power cord to a standard socket 220-230 V.
- ❖ Connect carefully the air supply hose to all outlets of cuff (**either arm, foot or waist**)
- ❖ Put the proper cuff to the proper place in order to make the therapy
- ❖ Turn on the device to start the therapy
- ❖ Choose the appropriate pressure and adjust the time of the therapy, with pushing the relevant button.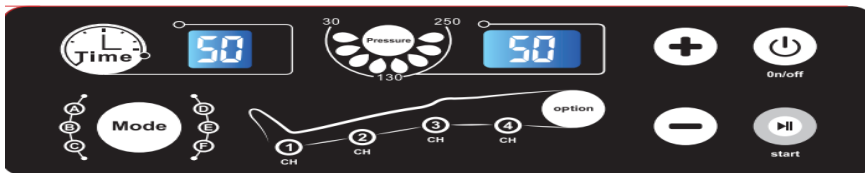

- ❖ Firstly turn off the pressure pump and then remove the cuff

## SPECIFICATIONS OF PRESSURE PUMP


SIZE	POWER	VOLTAGE - FREQUENCY	PRESSURE RANGE (mmHg)	PRESSURE THERAPY TIME
24 cm X 20 cm X 11cm (L X W X H)	30Watt	220Volt, 50Hz	30 – 240 mmHg	1 - 30 min


4-OUTLET DIGITAL PUMP for  
PRESSURE THERAPY  
**REF: 0806211**



## PROGRAMS FOR 4-OUTLET DIGITAL PUMP for PRESSURE THERAPY (REF: 0806211)

- ❖ Program A is for an effective relaxing massage.
- ❖ Program B is for lymphatic lump or prevention and treatment
- ❖ Program C is for massage and muscle relief.
- ❖ Program D partly contributes to slimming feet, arms and waist.
- ❖ Program E is for blood circulation and lymphatic congestion, is a combination of programs A + B
- ❖ Program F is a combination of programs B + C

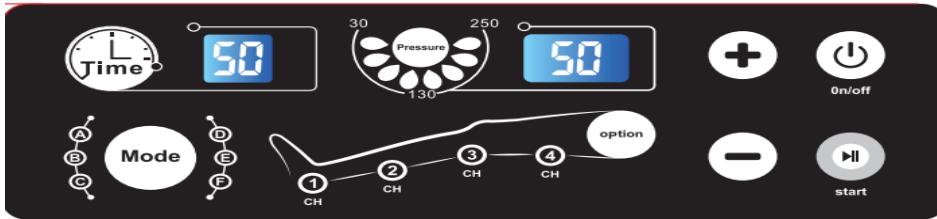### DEPICTION OF 6 AVAILABLE PROGRAMS
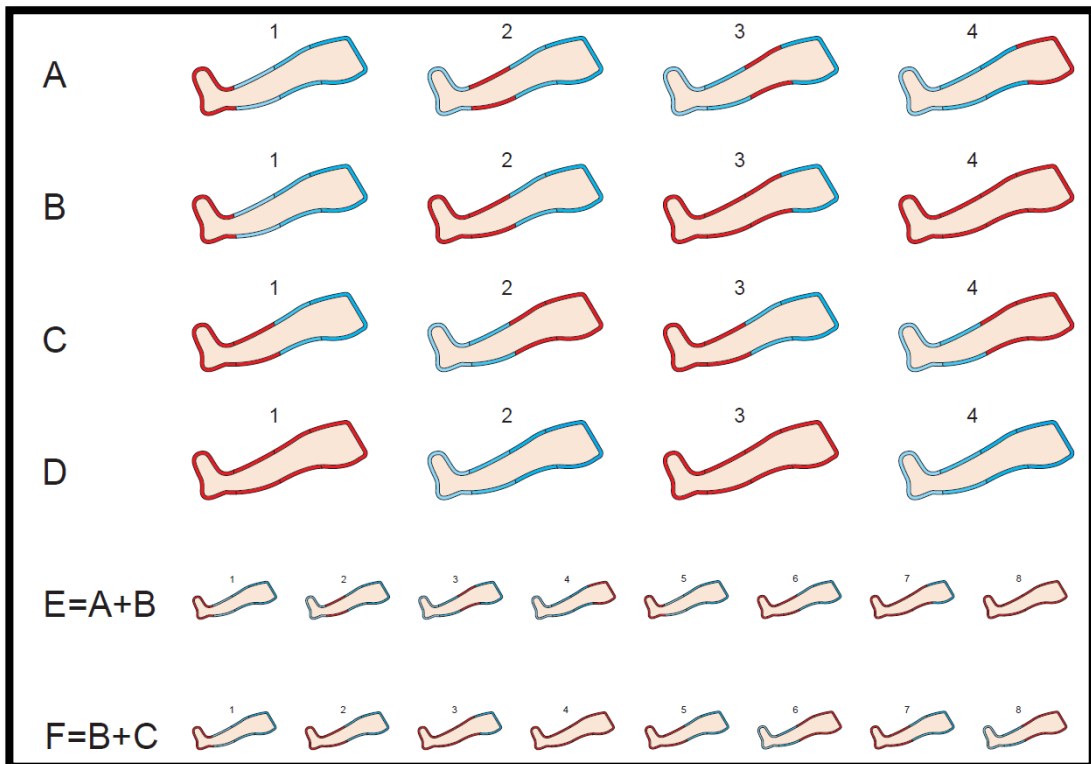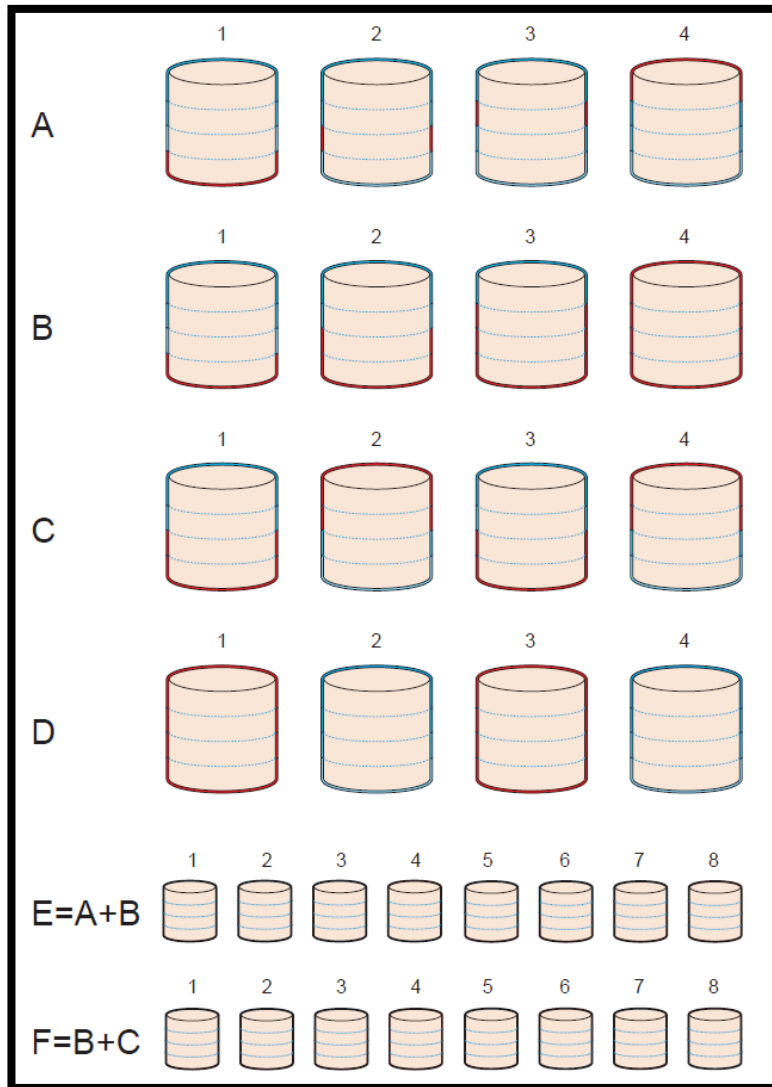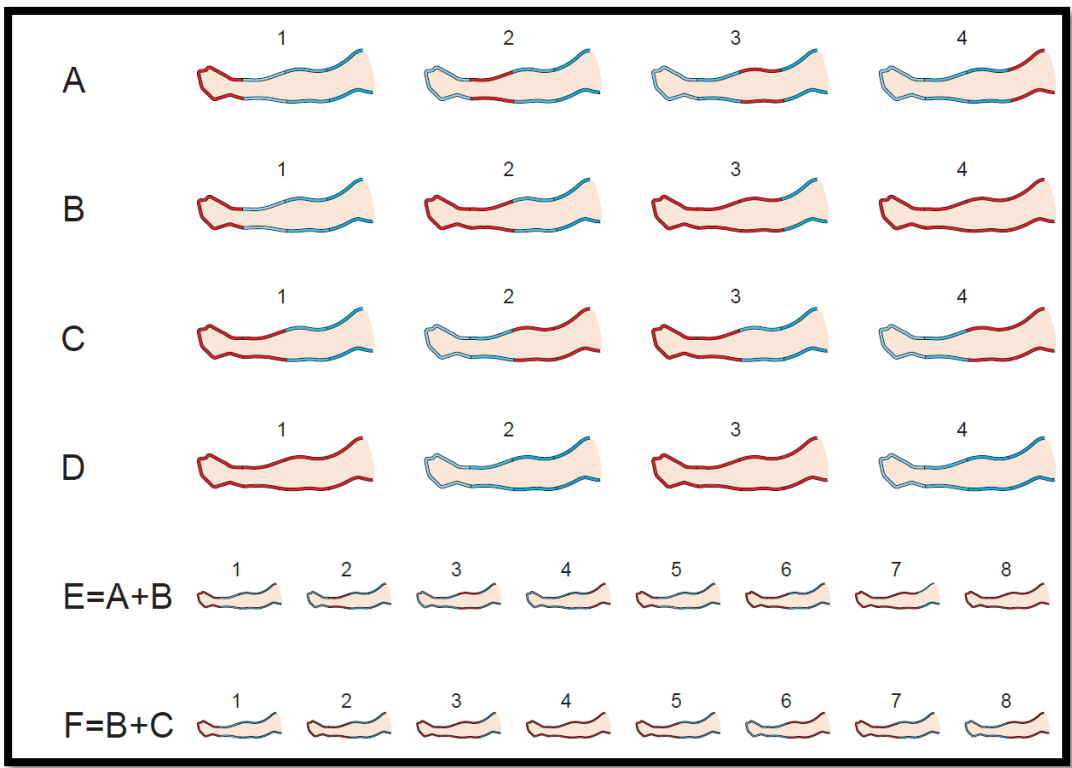
## CONNECTION OF TWO CUFFS IN THE SAME PUMP

Also there is a chance if it is essential to connect two cuffs in the same pump without problem in the therapy.

This connection is succeeded using the following part. (This part is not included in the package and it is available on request)

4–Outlet Connector of two cuffs  
**REF: 0806221**

The procedure of connection in the same pump is illustrated and described below:

1. Connect the part 3 with the part 2 of the connector.
2. Connect the part 4 with the part 2 of the connector.
3. Connect the part 1 of the connector with the pump socket.
4. Turn on the pump by having it connected both cuffs on the same pump

Make sure the color combinations of the wires go into the corresponding correct socket. That is, the light gray cap in the corresponding light gray slot and the dark gray cap in the corresponding dark gray slot, as shown below.

**CUFF TYPES FOR PRESSURE THERAPY**  
(Available separately from the device)

Leg cuff

Hand cuff

Waist cuff

## SIZES FOR CUFF TYPES

REF NUMBER	TYPE/SIZE	MEDIUM	LARGE	X-LARGE
0806213 (Medium) 0806214 (Large) 0806225 (X-Large)	Leg Cuff	<u>WIDTH</u> : 24 cm (W) <u>LENGTH</u> : 67 cm (L) <u>CIRCUMFERENCE</u> : 62 cm	<u>WIDTH</u> : 40 cm (W) <u>LENGTH</u> : 67 cm (L) <u>CIRCUMFERENCE</u> : 68 cm	<u>ΠΛΑΤΟΣ</u> : 40cm (W) <u>ΜΗΚΟΣ</u> : 86cm (L) <u>ΠΕΡΙΜΕΤΡΟΣ</u> : 70 cm
0806216 (Medium) 0806217(Large)	Hand cuff	<u>WIDTH</u> : 19cm (W) <u>LENGTH</u> : 87 cm (L)	<u>WIDTH</u> : 25 cm (W) <u>LENGTH</u> : 87 cm (L)	
0806218 (Medium) 0806219(Large)	Waist cuff	<u>WIDTH</u> : 24 cm (W) <u>LENGTH</u> : 116 cm (L)	<u>WIDTH</u> : 24 cm (W) <u>LENGTH</u> : 126 cm (L)	

## MAINTENANCE

- ❖ To clean the cuff use a soft cloth or sponge & water with mild soap.  
ATTENTION: Do not use acidic & caustic dilutions.
- ❖ Do not smoke & do not use any flame near the cuff.
- ❖ Do not have the cuff in contact with direct heat sources.
- ❖ If you need to save the cuff, empty of air, folding it
- ❖ Keep or store the pump in a clean, dry, well ventilated environment.
- ❖ Before cleaning the pump, disconnect from the electrical supply.
- ❖ Clean the pump with a dry cloth.  
ATTENTION: Do not bring the pump into contact with liquids.

## **IMPORTANT (!)**

### **CAUTIONS**

- ❖ Do not disconnect the hoses while the pump is running.
- ❖ Do not use sharp objects near the cuff.
- ❖ Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- ❖ Do not use abrasive alcohol-based while cleaning the cuff. This could damage the material of the layer.
- ❖ Clean the upholstery after each daily use.
- ❖ We recommend that you clean the upholstery with a mild detergent to protect it from oil stains and dust. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the surface and then wipe with a clean towel. The best solution is to be cleaned with warm water mixed with a mild soap and then thoroughly dried.
- ❖ Protect the parts from scratches, cuts and punctures.
- ❖ Do not leave the product run on, if not using it.

### **STORAGE**

- ❖ Switch off the device, unplug it from the power supply and disconnect the cuff from the pump.
- ❖ Place the cuff on a flat surface and roll or fold from the head end towards the foot end.
- ❖ Allow air slowly escape the cuff so we will reduce storage space
- ❖ Close the protective cap cover on the quick connector.
- ❖ Place cuff & the pump in the carrying bag (if available) and store it in a moisture-free place.

### **ATTENTION (!)**

- ❖ In case of damage or malfunction of the pump, please contact your authorized and trained dealer.
- ❖ Do not use the product for a purpose not indicated in this manual
- ❖ Only qualified personnel or trained for the treatment and prevention of bedsore, can operate with the device
- ❖ MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- ❖ MOBIAK S.A reserves the right to change the information contained in this document without previous notice.

## ENVIRONMENTAL PROTECTION

If you find one day that your appliance needs replacement, or no longer serves you, consider the Environment:

- 1). Do not dispose of your device along with the municipal waste (this is also the importance of the stated recycling symbol).
- 2). Contact your municipal authority to indicate the point of disposal of your device for recycling.
- 3). With the most useless your device in the right place  
Recycling helps to protect the environment and the operation again your device materials.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

## INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

## DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

---

**WARRANTY**

The product has warranty for two (2) years in pump & six (6) months in the cuff, from the purchase date.

The warranty covers factory defects & does NOT cover defects resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product instructions for use. It also does NOT cover worn and whatever it is going to be damaged during use or over time.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) and transportation companies are not covered by the warranty.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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