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INTENDED USE:

The bath board is designed to allow people with mobility problems to be able to bath while sitting on it, reducing muscle fatigue due to the standing position as well as the risks of slipping. It is adjustable & can be fixed to the edges of the bathtub and has a rotating base for extra comfort and ease of adjusting the position.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAC S.A reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$. Deviation for the Dimensions of the Products.

GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the product by any removable parts.
- Do not use the product for a different purpose from the intended one.
- Use extra caution on wet and slippery surfaces.
- Do not exceed the weight limit of the safe working load.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

BEFORE USE:

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- The edges are firmly touching the bathtub.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the parts to ensure the absolute safe use of the product.
- We do not recommend that the product is issued to anyone without first carrying out a full assessment of their specific needs. It is important that the user is given the necessary support by the product as prescribed by a qualified professional.

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

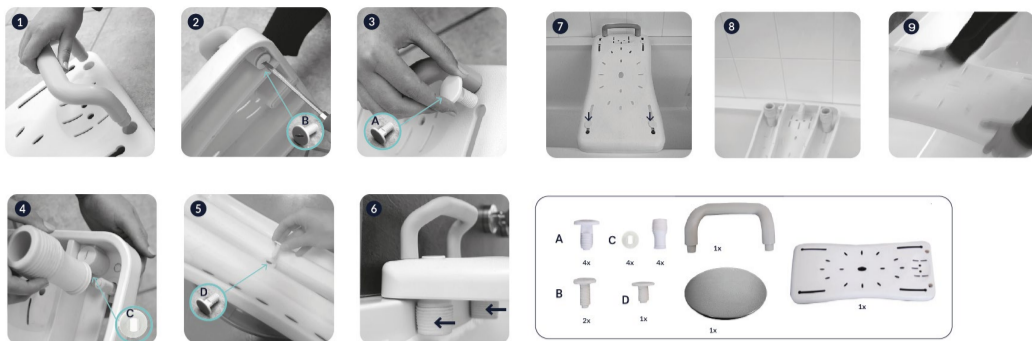
The user must make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate cleaning.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. DO NOT leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.

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WARRANTY:

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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FURTHER INSTRUCTIONS:

- Position the product on the bathtub and check that it fits in this position.
- Transfer the user in the product.
- Make sure that the product is secure and does not swing before starting the bath.
- Do not leave the user alone, unsupervised.

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INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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MOBIAK S.A