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**INTENDED USAGE:**

The product is designed as a bed aid and for back lifting of people who are in bed for a long time.

If you use it in any other way, it is likely to cause an accident or damage to the product.

**PRODUCT CHARACTERISTICS:**

Dimensions (LxW): 56 x 47cm.

5 different adjustment positions from 25 to 80 degrees.

With Height 34 from 60 cm

**SAFETY WARNINGS AND INDICATIONS:**

- The product is assembled, ready for use.
- The maximum user weight is 75 Kg.
- Before every use, be sure the product is correctly assembled in a stable position.
- The product can be used only on surface that are free from obstacles.
- The risk of finger injuries may increase during the mounting or assembling process.
- Do not modify the product and its components in any way.
- Do not use the product if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the product should be removed from the bed to avoid any obstruction.
- Do not use the product for a different purpose from the intended one.
- The product must not be stored outdoors.

**MAINTENANCE INFO:**

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

**REUSE REQUIREMENTS:**

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized and must be checked by qualified dealers.

**CORRECT DISPOSAL PROCEDURE:**

Do not dispose the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

**INCIDENT REPORTING:**

If there was a serious incident involving this product, contact the manufacturer of this product at [mobiakcarequality@mobiak.com](mailto:mobiakcarequality@mobiak.com)

**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

**WARRANTY:**

The product is guaranteed for a period of one (1) year from the purchasing date. The warranty covers factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery. Also DOES NOT cover damages or defects caused by natural disasters, unauthorized maintenance or repair and supply carriers problems.