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**INTENDED USE:**

The canes are walking aids that can be used with one hand and are aimed for people with limited mobility and difficulty for walking, for disabled or patients to help move in and out of the house, or/and their rehabilitation

**CAUTION (!):**

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- ΜΟΒΙΑΚ S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- ΜΟΒΙΑΚ S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

**GENERAL SAFETY WARNINGS:**

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.

**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

**BEFORE USE:**

1. Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame)
2. Check the correct assembly of the device.
3. Check if the safety washer is tightened well.

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

**MAINTENANCE, CLEANING & DISINFECTION INFORMATION:**

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user must make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. DO NOT leave the product wet.
6. If you need to disinfect the product, use a common, mild disinfectant.
7. Protect the item from scratches, cuts and punctures.

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**WARRANTY:**

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

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**ASSEMBLING:**

The Canes are delivered assembled & ready to use. To adjust the height, press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished.

**FURTHER INSTRUCTIONS:**

Hold the cane with the opposite hand to the weak leg and with the longer side of the support base directed toward the inside, if it is 3-leg or 4-leg cane.

Bring the product forward together with the foot of the weak leg loading the full weight on the stronger leg.

During walking, be careful not to place the device neither too close to the body to avoid tripping over one of tips nor too far away to avoid an imbalance.

In case of use on stairs, verify that the size of the step are sufficient to contain the base of device.

To go up the stairs, the user have to put first the cane on the stair, second to raise the stronger leg, and last the weaker leg. To go down the stairs with the cane must follow the injured leg, and finally the good leg. If necessary, get help from an assistant.

0812205	30 x 21 cm	74-97 cm	N/A	225 Kg
0812206	21 x 16 cm	74-97 cm	N/A	225 Kg

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**INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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USER MANUAL: BARIATRIC TETRAPOD CANE

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MOBIAK S.A