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**INTENDED USE:**

The heel protector is intended to reduce and redistribute pressure on the heel and ankle. Soft mesh fabric and silicone foam lining ensure breathable material properties and comfort for the user.

**CAUTION (!):**

- Read these instructions carefully before using the product. If you have any questions, consult a healthcare professional or the store where you bought it
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately.
- Product should be installed and fitted by a qualified healthcare professional.
- Although the product is not for single use, it should only be used by one patient. The product must not be reused by other patients.
- Use of this product for purposes other than those specified in this manual is prohibited.
- Always check the state of wear and tear of the parts, particularly the closure of the velcro, in order to ensure the device is used in total safety.
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

**DECLARATION OF CONFORMITY:**

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 2017/745 and where appropriate, the standards and legislation referred to.

**INCIDENT REPORTING:**

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

**GENERAL SAFETY WARNINGS:**

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- The fitting of the product must be done by qualified health personnel.
- For the product to function properly, must be regularly checked to ensure that it is in good condition, without signs of wear and tear.
- Clean the product regularly and maintain proper hygiene.
- Must be stored away from sharp or sharp objects to avoid puncturing or tearing

**BEFORE EVERY USE:**

- In case of damage do not use the product and contact your dealer for further instructions.
- Always check the condition of wear for the parts to ensure the absolute safe use.
- Regularly check that the straps are tense and in good condition (free of grease, cuts, tears, etc.) and that the velcro close properly.

**ENVIRONMENTAL PROTECTION:**

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

**MAINTENANCE, CLEANING & DISINFECTION INFORMATION:**

To guarantee a safe use and adequate standard of hygiene, the user should perform these cleaning procedures.

1. Clean the product frequently.
2. Remove the evident dirt before proceeding with the most accurate cleaning.
3. Close the velcro before starting the washing procedure.
4. Machine washable at 70°C.
5. It is recommended using a washing bag in the washing procedure.
6. Do not use bleach.
7. Do not iron.
8. Do not dry clean.
9. Do not tumble dry.
10. DO NOT leave the product wet.
11. If you need to disinfect the product, use a common, mild disinfectant.
12. After washing, make sure that the product is completely dry before use.
13. Store in temperatures between 0 - 40°C.

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The product is ready for use. It's fitting and application is the sole responsibility of the treating physician and must be done by him or by specialized health personnel.

1. Place the Heel Boot carefully on the user's foot and carefully close the velcro straps.
2. The section with cushion beneath the heel keeps the foot elevated to reduce pressure on the heel.
3. If user lay on the side, the product will evenly redistribute the weight and reduce pressure on the ankle.

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**WARRANTY:**

The product is guaranteed for a period of two (2) years from the purchasing date. The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by the dealer.

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USER MANUAL: HEEL PROTECTOR WITH CUSHION

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