EL EN

MULTILINGUAL
MANUALS

MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

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PRODUCT DESCRIPTION

EN

Anti-decubitus mattresses are specifically designed to treat and help bedsores in bedridden patients. They can be used for hospital or home use. Evenly distribute the pressure on the body to prevent edema.

The pump switches the air pressure in the mattress, creating a sense of light massage and activate the immobilized due sprawling cells.


This product is indicated and suggested for patients who are likely to spend more than 15 hours a day in bed and for patients with medium to high risk of developing ulcers & bedsores (Stage II to III)

OPERATION USE

For the stripped mattress

- Remove the foam layer from the "roll" package and leave it for at least 12 hours to return to its normal form.
- Place the mattress in the special tray under the air mattress.
- The special case opens with a zipper around the air mattress.
- Place the air mattress over the bed mattress, having first stretch the bottom sheet, making sure the air inlet is at the side of the legs.
- Place a clean, dry sheet over the air mattress.
- Attach the mattress to the pump using the appropriate tubes in the package, when you hear the "click", it means that they are properly fitted & secured.
- Connect the pump to the power supply, press the **ON / OFF** switch, and turn the switch to the maximum to inflate the airfoil BEFORE used by the patient
- When the airway is fully inflated you can adjust the pressure range, depending on the user requirements, using the switch

For Air Pump: DESCRIPTION DISPLAY / FUNCTION

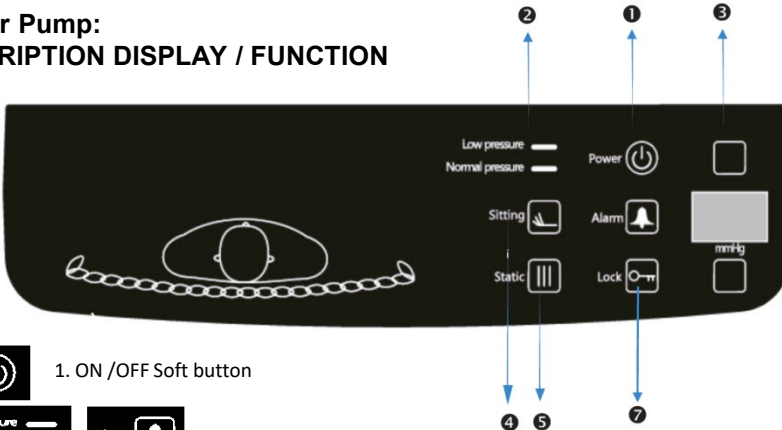
1. ON /OFF Soft button

2. LOW PRESSURE

If the air pressure is lower than the specified level, an alarm can be triggered where it can be recalled

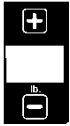3. AUTOMATIC PRESSURE CONTROL

Display of pressure range adjustment in kg of patient (30-180 kg.).
Hold down the key to jump fast, each step is 10kg.

4. SEAT OPERATION

The display shows the letter "L"
It offers extra support to patients when they are seated.

5. STATIC OPERATION

Both zones inflate together, no pressure switching, LED display shows "- -" for 30 minutes and then switch back to Alternating Pressure.
Use only for nursing treatment.

4. In the event of a power failure, an alarm will sound, press the "ALARM" key to mute until the power returns.


5. By pressing it, it locks all functions of the device.
To switch it off or on, hold it down for 5 seconds continuously.

SPARE PARTS

| REF. NUMBER | DESCRIPTION |
|-------------|---|
| 0806289 | Anti-Decubitus STRIPE Mattress 5"+3" FOAM Mattress |
| 0806290 | Pump for Anti-Decubitus STRIPE Mattress 5"+3" FOAM Mattress |


Front

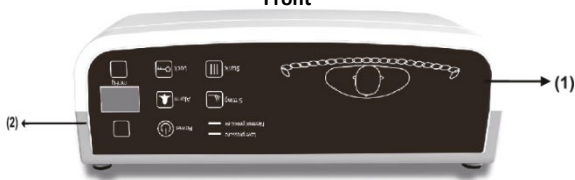
1. Control Panel
2. Quick Connection female


3. Quick connection male
4. Fuse
5. Filter
6. Filter cover
7. Bed hooks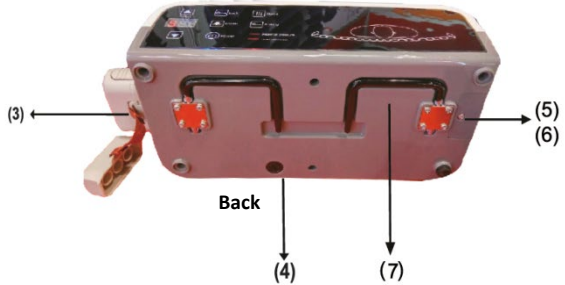

INSTALLATION

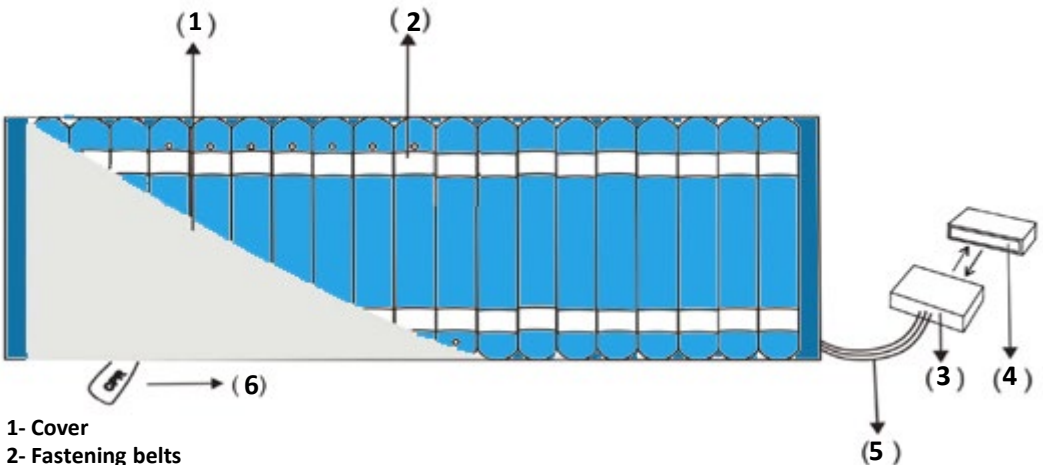
- 1- Cover
- 2- Fastening belts
- 3- Quick connection male
- 4- Quick connection cover
- 5- Air supply pipes
- 6- CPR valve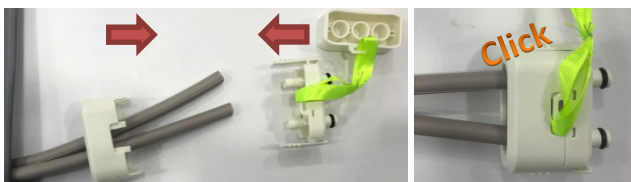

- Place the mattress directly on the bed frame with the tubes going to the pump facing downwards.
- Hang the air cushion pump on the bed where the legs end up or place it on a firm flat surface.
- Connect the quick links as shown in the pictures left.

- Connect the male part of the quick connection to the female part where it is located on the pump.
- Next, check the air supply hoses so that they are not beneath the mattress or that they are not folded to a degree of air pass difficulty.
- After the final check, connect the power supply to the wall power supply.
- Turn on the device.
- We wait about 12 minutes for the airbag to reach its maximum air capacity. Once this is done, the Normal Pressure light will turn green. The Low Pressure lamp will blink for the entire duration of this inflation.

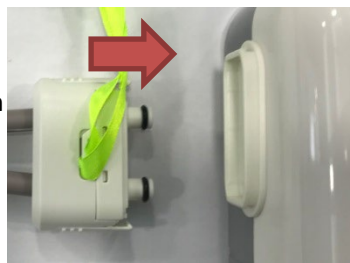NOTICE

If Cardio-Pulmonary Resuscitation (CPR) is required in the patient, immediately pull the valve (CPR) to release air directly from the mattress.

The valve (CPR) is located on the right side at the top of the mattress.

Also remove the connection from the pump for an even faster deflation of the mattress.

SPECIFICATIONS

| | | |
|-----------------|---------------------------------------|--|
| PUMP | Operating Voltage (V) -Frequency (Hz) | 220 VAC/ 50HZ |
| | Air outlet | ≥ 7 liters /min |
| | Pressure range | 30mmHg – 60mmHg |
| | Rotation cycle timing | 12 min |
| | Power | ≤ 7 W ± 10% |
| | Noise | ≤ 30 dB ± 10% |
| | Dimension (MxΠxY) | 27,5 x 13,5 x 9 cm |
| | Pump weight | 2 Kg |
| | Maximum inflation time | Approximately 12 minutes |
| | User interface type | All Keys are Touch, only the On / Off button is soft touch |
| MATTRESS | Max patient weight | 155 Kg |
| | Color | Blue |
| | Material | Nylon TPU & Stretchable PU (cover) |
| | Air Tube Height (inflated) | 20 cm |
| | Mattress Height | 0,22 mm |
| | Dimensions | 200 x 90 cm (18 cells) |
| | Height of foam mattress | 8 cm |
| | Connection tube length | 150cm |
| | Weight | 10 Kg |

STORAGE

- Switch off the device, unplug it from the power supply and disconnect the air-mattress from the pump.
- Place the air-mattress on a flat surface and roll or fold from the head end towards the foot end.
- Allow air slowly escape the mattress so we will reduce storage space
- Close the protective cap cover on the quick connector.
- Place air-mattress & the pump in the carrying bag (if available) and store it in a moisture-free place.

MAINTENANCE

- To clean the mattress, use a soft cloth or sponge & water with mild soap. ATTENTION: Do not use acidic & caustic dilutions.
- If you need to store the mattress, unplug it from the power supply and disconnect the air-mattress from the pump. Empty the air and fold it.
- Keep or store the pump in a clean, dry, well-ventilated environment.
- Before you clean the pump, disconnect from the power supply.
- Clean the pump with a dry cloth. Do not bring the pump in contact with liquids.
- Clean the product after each daily use. We recommend that you clean it with a mild detergent to protect it from oil and dust stains. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the product & then wipe it with a clean towel. The best solution is to clean it with warm water mixed with mild soap and then wipe it well.

IMPORTANT (!)

CAUTIONS

- Do not disconnect the hoses while the pump is running.
- Do not use sharp objects near the mattress.
- Do not exceed the maximum load (kg) recommended by the manufacturer
- Do not use, while bathing the patient on the bed.
- Protect the product by oils, lotions and creams, by covering it with the cotton wool fabric, or a bath towel.
- Do not use abrasive alcohol-based cleaning when the mattress. This could damage the material of the layer.
- Protect the layer from scratches, cuts and punctures.
- Do not leave the product run on, If not using it.
- Do not smoke & do not use any flame near the mattress.
- Do not have the mattress in contact with direct heat sources.

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained dealer.

CAUTION! (!)

Do not use the product for a purpose not indicated in this manual

Only qualified personnel or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAC S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration of the product.

MOBIAC S.A reserves the right to change the information contained in this document without previous notice.

EC Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

NORTON SCALE DETAILS

The Norton Scale was developed in the 1960s and is widely used to assess the risk for pressure ulcer in adult patients. The five subscale scores of the Norton Scale are added together for a total score that ranges from 5-20. A lower Norton score indicates higher levels of risk for pressure ulcer development.

| SCORE | RISK FOR PRESSURE ULCER DEVELOPMENT |
|-------|-------------------------------------|
| >18 | LOW |
| 18-14 | MEDIUM |
| 14-10 | HIGH |
| < 10 | VERY HIGH |

Please be noted that the Norton Scale is a tool only to be used by trained healthcare professionals, and only as a supplement to a thorough patient state evaluation.

| MBK CODE | SUITABLE FOR NORTON SCALE SCORE |
|----------|---------------------------------|
| 0806288 | 15 – Medium Risk |

WARRANTY

The product has warranty for two (2) years in pump and six (6) months on the mattress and the other non-mechanical parts, from the purchasing date. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems and transportation companies are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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