EL EN

MULTILINGUAL
MANUALS

MOBIAK®

MEDICAL • GASES • FIREFIGHTING est. 1977

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PRODUCT DESCRIPTION

Anti-decubitus mattresses are specifically designed to treat and help bedsores in bedridden patients. They can be used for hospital or home care use. Equally distribute the pressure on the body to prevent edema.

The pump switches the air pressure in the mattress, feeling a sense of light massage and activate the immobilized due sprawling cells.

This product is indicated and suggested for patients who are spend more than 15 hours a day in bed and for patients with medium to high-risk development in ulcers & bedsores (Stage II to III)


OPERATION USE

For the mattress

- Stretch the bottom sheet and place the air mattress on the bed mattress, make sure the air inlet is on the leg side.
- Secure it with fixing straps below the mattress
- Place on the air mattress a thin breathable fabric, if not accompanied by its own cover, so the patient does not come in direct contact with it.


For the air pump

- Attach the mattress to the pump using the appropriate tubes in the package. When you hear the "click", it means they are properly fitted & secured.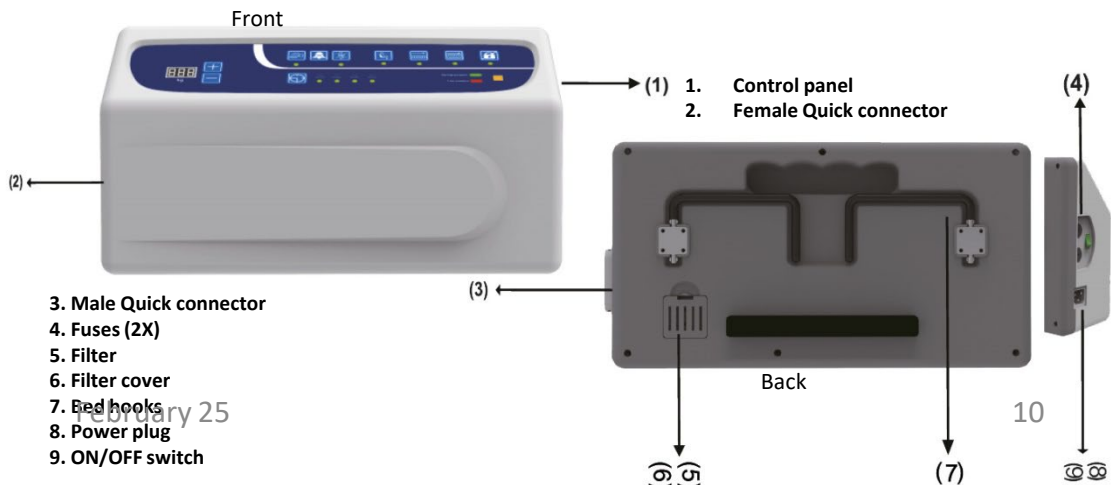

FOR THE PUMP

FUNCTIONS INDICATIONS / DESCRIPTIONS

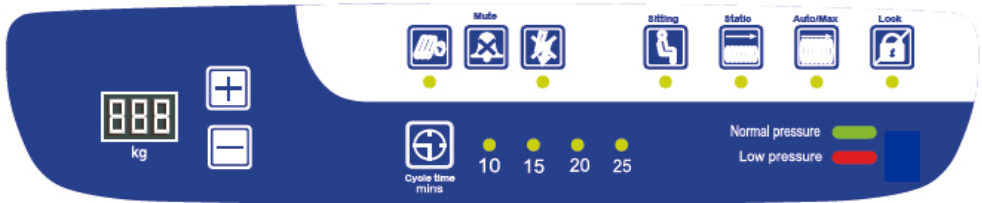1 AUTO PRESSURE ADJUSTMENT

The pressure indication range adjustment in kg of patient (30-200 kg).
Press the button to jump fast, each step is 10kg.

2 CYCLE TIMING

Set the rotation cycle to 10, 15, 20, 25 minutes in alternating pressure mode.

3 LOW PRESSURE

The Low Pressure LED will illuminate the sound alarm when the pressure is lower than the settings of the pump.
Press the "Mute" button to sound off the alarm.

4 POWER LOSS

If there is a power failure, the alarm will sound on.
Press "Mute" button when the LED turns off, the alarm goes off.

5 LEAKAGE ALARM

If the mattress constantly loses pressure, the alarm turns on.

Press the "Mute" button to reset

The alarm sound turns off ends when the LED is off.

6 SEAT/STATIC / AUTO FIRM

a. SEAT OPERATION

Supports the patient in a seated position.
Turns back to alternating mode after 30 minutes.

b. STATIC OPERATION

Provides pressure at both tubes.
Turns back to alternating mode after 30 minutes.

c. AUTOFIRM OPERATION

It provides pressure at both tubes and inflates the mattress to its maximum capacity very fast.
Turns back to alternating mode after 30 minutes.


7 LOCK


Press the button for 2 second to lock all functions of the device.

To switch it off, press the button for 3 seconds.

INSTALLATION

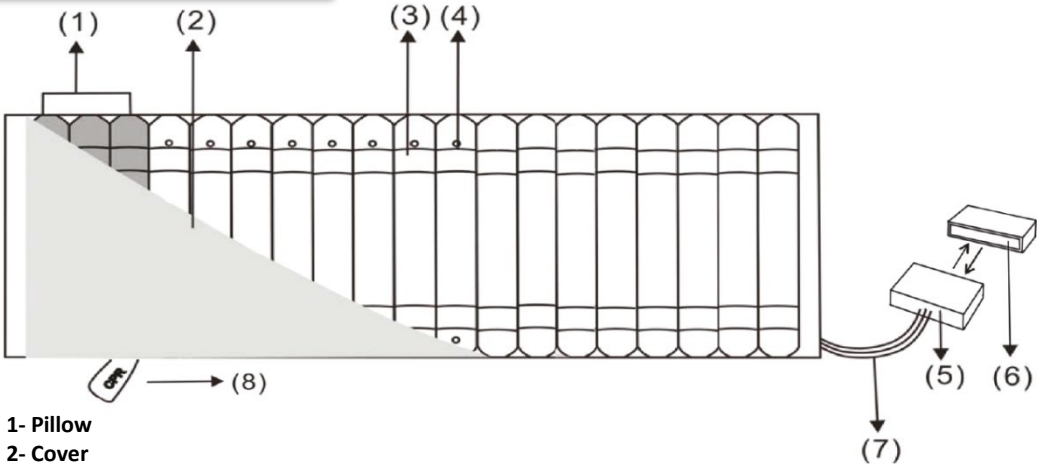
- 1- Pillow
- 2- Cover
- 3- Fastened belts
- 4- Air Holes (Six of the aerosol rods have vent holes at their ends).
- 5- Quick connection male
- 6- Quick connection cover
- 7- Air supply pipes
- 8- CPR valve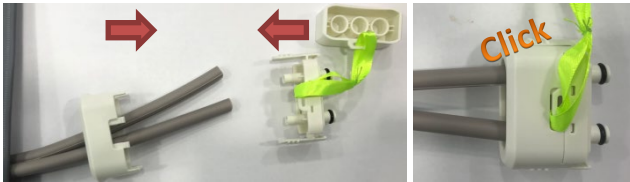

- Place the mattress on the bed frame with the pump tubes on the leg side.
- Hang the pump on the leg side as well or on a flat surface.
- Connect the quick links as shown in the picture.

- Connect the male part of the quick connection to the female part where it is located on the pump.
- Make sure the air supply pipes are not under the mattress or folded so the air can flow in properly.
- After the final check, connect the power supply to the power.
- Turn on the device.

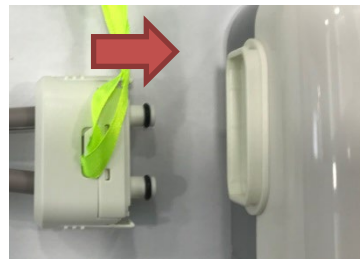If Cardiac Rejuvenation (CPR) is required in the patient, immediately pull the valve (CPR) to release air directly from the mattress.

The valve (CPR) is located on the right side at the top of the mattress.

Also remove the connection from the pump for an even faster deflation of the mattress.

SPECIFICATIONS

PUMP	Operating Voltage (V) -Frequency (Hz)	220 VAC/ 50HZ
	Air outlet	≥ 12-14 liters /min
	Pressure range	22mmHg – 55mmHg
	Rotation cycle timing	10/ 15/ 20/ 25 min
	Power	≤ 14 W ± 10%
	Noise	≤ 30 dB ± 10%
	Dimension (LxWxH)	34,5 x 19 x 14,5 cm
Pump weight	3,5Kg	
MATTRESS	Max patient weight	200Kg
	Color	Blue
	Material	Nylon TPU & Stretchable PU (cover)
	Air Tube Height (inflated)	20 cm
	Mattress Height	0,22 mm
	Dimensions	200 x 90 cm (20 cells)
	Connection tube length	150cm
	Weight	10 Kg

<u>REF. NUMBER</u>	<u>DESCRIPTION</u>
0806294	TUBULAR MATTRESS WITH VENTILATION & CPR
0806295	PUMP ICU

STORAGE

- Switch off the device, unplug it from the power supply and disconnect the air-mattress from the pump.
- Place the air-mattress on a flat surface and roll or fold.
- Allow air slowly escape the mattress so we will reduce storage space
- Close the protective cap cover on the quick connector.
- Place air-mattress & the pump in the carrying bag (if available) and store it in a moisture-free place.

MAINTENANCE

- To clean the mattress, use a soft cloth or sponge & water with mild soap. ATTENTION: Do not use acidic & caustic dilutions.
- If you need to store the mattress, unplug it from the power supply and disconnect the air-mattress from the pump. Empty the air and fold it.
- Keep or store the pump in a clean, dry, well-ventilated environment.
- Before you clean the pump, disconnect from the power supply.
- Clean the pump with a dry cloth. Do not bring the pump in contact with liquids.
- Clean the product after each daily use. We recommend that you clean it with a mild detergent to protect it from oil and dust stains. A common practice is to use a spray bottle filled with a mixture of anti-bacterial soap and water. Spray the product & then wipe it with a clean towel. The best solution is to clean it with warm water mixed with mild soap and then wipe it well.
- Do not smoke & do not use any flame near the mattress.
- Do not have the mattress in contact with direct heat sources.

IMPORTANT (!)

CAUTIONS

- Do not smoke & do not use any flame near the mattress.
- Do not expose the mattress to direct heat sources.
- Do not disconnect the pipes while the pump is on.
- Do not use sharp objects near the mattress.
- Do not exceed the maximum load (kg) recommended by the manufacturer.
- Do not use when bathing the patient.
- Protect the product of oils, lotions and creams, by covering it with cotton wool fabric, or a bath towel.

IMPORTANT (!)**CAUTIONS**

- Protect the layer from scratches, cuts and punctures.
- Do not leave the product on, If it is not in use.

IMPORTANT (!)

In case of damage or malfunction of the pump, please contact your authorized and trained dealer.

CAUTION! (!)

Do not use the product for a purpose not indicated in this manual

Only qualified personnel or trained for the treatment and prevention of bedsores, can operate with the device

MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.

MOBIAK S.A reserves the right to change the information contained in this document without previous notice.

EC Declaration of Conformity

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

ENVIRONMENTAL PROTECTION

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

NORTON SCALE DETAILS

The Norton Scale was developed in the 1960s and is widely used to assess the risk for pressure ulcer in adult patients. The five subscale scores of the Norton Scale are added together for a total score that ranges from 5-20. A lower Norton score indicates higher levels of risk for pressure ulcer development.

SCORE	RISK FOR PRESSURE ULCER DEVELOPMENT
>18	LOW
18-14	MEDIUM
14-10	HIGH
< 10	VERY HIGH

Please be noted that the Norton Scale is a tool only to be used by trained healthcare professionals, and only as a supplement to a thorough patient state evaluation.

MBK CODE	SUITABLE FOR NORTON SCALE SCORE
0806293	13 – High Risk

The product has warranty for two (2) years in pump and six (6) months on the mattress and the other non-mechanical parts, from the purchasing date. The warranty covers factory defects and not damages resulting from improper use / maintenance of the product.

The warranty apply factory defects & DOES NOT apply damages resulting from improper use , poor maintenance, modification, abuse or non-compliance with the product's operating instructions.

Damage or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems and transportation companies are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user- customer or by dealer.

Buyers Data

Name:

Date of purchase:S/N.....

LOT:.....Phone:

Resellers Data

Name :

.....

.....

Address:.....

.....

Phone:

Reseller Stamp

INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

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Description of symbols that have been printed on the label, Master Carton and or User Manual

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