**GR EN**

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## INTENDED USE:

The Crutches are walking aids that can be used with one hand and are aimed for people with limited mobility and difficulty for walking, for disable or patients, to help move in and out of the house or and their rehabilitation.

It is recommended to use two pieces at the same time, one in each hand.

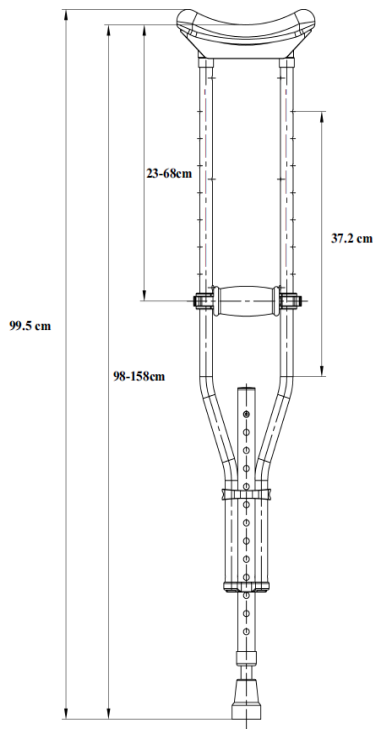
They can also be used if an injury or surgery requires you to protect your foot from being loaded with your weight.

REF:	FRAME	SIZE	ADJUST. HEIGHT	MAX LOAD	TUBE THICKNESS
<u>0804594</u>	ALUMINIUM	UNIVERSAL ( FITS SIZES SMALL, MEDIUM, LARGE)	98-158cm	136KG	1.15 mm

MASTER CARTON DIMENSIONS	PAIRS PER CARTON	N.W	G.W
26 x 10 x 53cm	1 PAIR	1.92KG	2.42KG

## GENERAL SAFETY WARNINGS:

- If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Keep away the product from heat sources.
- Save this document for future reference.
- Be careful when children are nearby and do not allow the children to play with the product.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not use the product for a different purpose from the intended one.
- Replace Crutch tip if it becomes worn.
- Keep handgrip wing nut tightened.
- Use extra caution on wet and slippery surfaces.
- Do not exceed the weight limit of the safe working load.

**ASSEMBLING:**

Open the package and take out the product.

Install the handlegrip to the armpit tube of the crutch

Align the red part with the hole position of the tube and press it in.

Connect the upper and lower part of the crutch by connecting the corresponding pipes.

The product is ready to use.  
To adjust the height, press the push button and slide the Crutch to the desired height. Be sure the button is fully protruded through the desired hole when finished.

## **BEFORE EVERY USE:**

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the parts to ensure the absolute safe use of the product.

## **FURTHER INSTRUCTIONS:**

1. The upper part of the crutches should be between 2.5 and 3.5 cm below your armpits while standing.
2. The Crutches handles should be at the height of the patient's hips.
3. The elbows should be slightly bent when holding the handles.
4. Hold the upper part of the crutches tightly to your side and use your hands to absorb your weight. Do not let the upper part of the crutches press against your armpits.
5. Lean forward slightly and place the Crutches about 35 cm in front of you. Start your step as if you were going to use your injured leg but throw your weight on the Crutches instead of your injured leg. Your body should swing forward. Complete your step normally with your uninjured foot. When your uninjured foot is on the ground, move the Crutches forward as part of the preparation for the next step. Focus on the ground, not your feet.
6. For the stairs we follow three simple rules: We go up or down one step at a time. When we go up: The uninjured leg goes up first after the injured and finally the crutches (but on the same step). When we go down: the crutches are placed on the next step followed by the injured leg and finally the uninjured leg (all on the same step)

## **CAUTION (!):**

In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.

- Do not use the product for a purpose not indicated in this manual
- MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- MOBIAK S.A reserves the right to change the information contained in this document without previous notice
- There is a  $\pm 3\%$ . Deviation for the Dimensions of the Products.

## MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate cleaning.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and cleancloth.
5. DO NOT leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.

## ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

## DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

## INCIDENT REPORTING:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

## **WARRANTY:**

The product has a warranty of good operation for two (2) years from purchase date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time.

Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product.

Shipping costs for goods of warranty are to be paid by the end user-customer or by the dealer.

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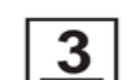
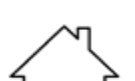
Description of symbols that have been printed on the label, Master Carton and or User Manual

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**MOBIAC S.A**